

# INSTRU-NEU

## CONCENTRATED NEUTRAL CLEANER

Instru-Neu, Instrument Cleaner Concentrate, is a highly concentrated liquid cleaner designed for the cleaning of surgical instruments prior to autoclaving.

Instru-Neu is also excellent for cleaning equipment and carts.

### DIRECTIONS FOR USE

Use Instru-Neu, Instrument Cleaner Concentrate, at a dilution of 1:128 (1 ounce of Instru-Neu to each gallon of water) for cleaning of surgical instruments. Instruments should be rinsed with potable water prior to autoclaving.

### FOR AUTOMATIC WASHERS

Use Instru-Neu, Instrument Cleaner Concentrate, at a dilution of 1:128 (1 ounce of Instru-Neu to each gallon of water) when cleaning instruments and carts in an automatic scrubber.

## TECHNICAL DATA

**APPEARANCE:** ..... Transparent Yellow Liquid

**ODOR:** ..... Light Lemon

**NON-VOLATILE MATERIAL:** ..... 25% Minimum

**DETERGENT TYPE:** ..... Synthetic - Biodegradable

**PHOSPHATE CONTENT:** ..... NONE

**DETERGENT CONTENT OF RECOMMENDED USE SOLUTION:** ..... 975 ppm ± 25 ppm

**FREE ALKALI:** ..... NONE

**FLASH POINT:** ..... NONE

**pH 1:128 in DEIONIZED WATER:** ..... 8.5 – 9.2

**pH of CONCENTRATE:** ..... 7.5 – 8.5



325 Leffingwell Avenue • Kirkwood, Missouri

[www.wexfordlabs.com](http://www.wexfordlabs.com)

(800) 506-1146